

LUNEAU Cochet and Bonnet AESTHESIOMETER

Instructions for cleaning and disinfection.

1. **Cleaning** with an alkaline detergent :

immerse the tip of the thread in an enzymatic solution of detergent-disinfectant ALKAZYME (or ANIOSYME) for a minimum of 5 minutes.

Rinse with water.

2. **Disinfection** , by immersion in a solution of glutaraldehyde ALKACIDE (or STERANIOS) during 15 minutes:

Following immersion, rinse thoroughly with clean water.

3. Cleaning of the black cylinder with disinfection wipes: ALKAPHARM WIPES or SEPTALKAN WIPES.